

# Loading Calibration for the AVANT ARC, REM or REM<sup>+</sup> Unit

---

The calibration values for the probe microphone(s) must be entered into your AVANT REM Software manually.

## Open the AVANT REM software

- Click the **FUNCTION** tab
- Click the **CALIBRATE** tab
- Click the **LOAD CALIBRATION FILE** tab
- Click the **MANUAL INPUT** tab
- Select **LEFT** (blue) **or RIGHT** (red) **probe tab**.

- **Enter the values on the calibration sticker, attached to the probe microphone cable, into the Reference and Probe microphone dB offsets.**

*If the value is a negative figure, you must enter the minus sign (-).  
If the value is a decimal number, you must enter the period (.)*

- Click **SET**
- If the values displayed are correct, click the **YES** tab.

When you are finished entering left, right or both probe microphone dB offsets, click **CLOSE**.

**MedRx**<sup>®</sup>

[www.medrx-int.com](http://www.medrx-int.com) • Sickingenstr. 70-71, 10553 Berlin, Germany • +49 30 7071 4650 • [medrx-sales@maico.biz](mailto:medrx-sales@maico.biz)

---